

In-service checklist

Large bone power tool guide for O.R. staff



The points below should be covered when carrying out a large bone power tool in-service for scrub personnel.

Tips: To ensure a good turnout, before your in-service, advertise the important details in common places (above scrub sinks, bulletin boards, etc.) and via email.

Beginning an in-service by addressing difficulties and listing them in front of your participants is a best practice among trainers. From there, the in-service provider can walk through the steps necessary to resolve

these difficulties and cover any other relevant product information. To establish further understanding of the product, it is recommended that the staff be asked to perform functions on the equipment without assistance. It may also be beneficial to partner the group up and ask for a return demonstration.

To assure optimal participation and information absorption, it is recommended that in-services are made mandatory, attendance is tracked and followed up with a quiz or a test.

Handpieces

- Identify each type of power tool and explain what their action is and where they may be used.
- Name each attachment and give an example when it may be utilized.
- Explain how to activate and deactivate the safety switch on the handpieces.
- Demonstrate how to safely insert and remove an attachment from a handpiece.
- Show how to insert and remove the different saw blades safely.
- Demonstrate the operation and functions of the triggers on the handpieces.
- Explain when to use and demonstrate how to effectively use the pin and wire collets.

Batteries

- Identify the different types of batteries and explain what handpieces they accommodate.
- Demonstrate how to insert and remove batteries from the handpieces.
- If using aseptic batteries demonstrate how to transfer batteries into the battery casing using the transfer shield.
- Explain the fuel gauge indicator light on the battery.

In-service checklist

Large bone power tool guide for O.R. staff



Battery charger

- Show how to turn on the charger and insert and remove batteries from the charger terminals.
- Identify the key functions of the information screen.
- Explain the various indicator lights and what they mean.

References:

- 9100-001-330 Rev B Powertools Cleaning & Sterilization
- 7200-001-700 Rev D System 7 Care Instructions IFU
- 7202-001-720 Rev None EZOut System IFU
- 7203-000-000 Rev C System 7 Single Trigger Rotary IFU
- 7203-001-702 Rev B System 6 and System 7 Rotary Handpiece Attachments IFU
- 7205-001-700 Rev C System 7 Dual Trigger Rotary IFU
- 7206-001-700 Rev C System 7 Reciprocating Saw IFU
- 7207-001-700 Rev C System 7 Sternum Saw IFU
- 7208-001-700 Rev C System 7 Sagittal Saw IFU
- 7208-008-700 Rev B System 7 V-Notch Sagittal Saw IFU
- 7209-001-007 Rev D System 7 Precision Saw IFU
- 7209-009-700 Rev C System 7 High Speed Precision Saw IFU
- 7110-120-700 Rev C Universal Battery Charger IFU
- 7215-001-700 Rev D SmartLife Battery IFU

This document is intended to be a supplement to the product IFU. For detailed instructions, please consult the IFU.